



EuroSept®Plus Thermo Powder

- Alkaline powder for mechanical reprocessing of dental instruments
- Very good cleaning power
- Gentle on materials
- Miele and Melag approved



PRODUCT DESCRIPTION:

Henry Schein selected a cleaner liquid, a neutraliser and a rinsing liquid to accompany the Alkine powder for the perfect mechanical instrument cleaning cycle for by using thermodisinfectors. The liquids need to be added to their respective containers for automatic dosing during the cycle. The powder has to be added manually. EuroSept®Plus products are accepted by the major manufacturers of the thermodisinfectors.

Cleaner (granules) for mechanical reprocessing of dental instruments with the exception of handpieces and contra-angle handpieces.

After the cleaning step, neutralisation with Thermo Neutralizer is recommended. For spot-free drying of the instrument set, the rinse agent Rinse is necessary.

Instruments made of anodised aluminium should be tested for alkali resistance Labelling according to regulation (EC) 648/2004: > 30% phosphates, < 5% non-ionic surfactants.

CONCENTRATION:

0.3% = place 30g powder per cleaning cycle in the dosing flap (= two level measuring spoons)

COMPOSITION:

Corrosion inhibitors

SPECIAL INFORMATION:

Can causes skin irritation and serious eye irritation. Wear protective gloves/ eye protection. IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/ attention. Protect from frost, heat and sunlight. Store in original container at room temperature.

AVAILABLE DELIVERY FORMS:

REF 988-2722 6kg bucket

ALSO AVAILABLE:

- REF 988-2721 EuroSept®Plus Thermo Rinse
- REF 988-2723 EuroSept®Plus Thermo Neutralizer
- REF 988-2724 EuroSept®Plus Thermo Cleanser