



SAFESEPT^{MAX} Surface Disinfection

Rapid Surface Disinfection

- alcohol based
- residue-free
- fast acting
- avoids any discolorations of surfaces



PRODUCT DESCRIPTION:

Ready-to-use, aldehyde-free rapid disinfectant based on ethanol for an effective disinfection of medical inventory, medical devices as well as other kinds of surfaces. Especially suitable for difficult to reach surfaces. The aldehyde-free formulation avoids any discolorations of surfaces after the use of SAFESEPT MAX® Surface Disinfection.

Do not use on alcohol-sensitive surfaces (e.g. acrylic glass).

AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): quick disinfection and cleaning of alcoholresistant, non-invasive medical devices.

Acc. to Biocidal Products Regulation (BPR): quick disinfection and cleaning of alcohol-resistant surfaces of all kinds.

SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection), bactericide, yeasticidal as per VAH: 1 min.; tuberculocide (*M. terrae*): 1 min; limited virucide (including HBV, HIV, HCV), rotaviruses: 30 sec.; noroviruses (MNV): 1 min

APPLICATION:


Spray surfaces and objects with SAFESEPT MAX® Surface Disinfection from a distance of at least 30 cm until entirely moistened. After exposure time, wipe off surfaces with a clean single-use cloth if required. To avoid the formation of alcoholic fumes, SAFESEPT MAX® Surface Disinfection can be sprayed on a disposable tissue and used for wiping of surfaces by respective wetting. SAFESEPT MAX® Surface Disinfection does not leave any residues on surfaces.

COMPOSITION:

100 g ready to use solution contains: 45 g ethanol (CAS: 64-17-5).

SPECIAL INFORMATION:

Flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour. Use only outdoors or in a wellventilated area. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection. For Professional Use Only.

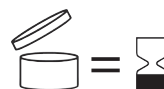



SAFESEPT^{MAX} Surface Disinfection

Rapid Surface Disinfection

- alcohol based
- residue-free
- fast acting
- avoids any discolorations of surfaces



PRODUCT STATUS/LISTINGS:

- Conform to the EU Biocidal Product Regulation (BPR, 528/2012).
- Conform to the EU guideline 93/42/EC for medical devices.

AVAILABLE DELIVERY FORMS:

- REF **988-2740** 1L Spray bottle
- REF **988-2741** 5L Canister

AVAILABLE ACCESSORIES:

- REF **979-6277** Sprayhead for 1 Liter Bottle
- REF **979-6276** Tab/dosing device for 5L canister
- REF **979-6278** Canister-key
- REF **979-6272** Dosing pump for 5L Canister