



# SAFESEPT MAX<sup>®</sup> WOUND SPRAY

## Wound Spray



### PRODUCT DESCRIPTION:

SAFESEPT MAX<sup>®</sup> wound spray is indicated for cleaning, moistening, antimicrobial decontamination and infection prevention in the wound by helping to remove biofilms and fibrinoid residues. In addition SAFESEPT MAX<sup>®</sup> wound spray can aid the gentle removal of dressings. The preservative (sodium hypochlorite) contained in SAFESEPT MAX<sup>®</sup> wound spray decontaminates a broad spectrum of bacteria (incl. pseudomonas aeruginosa, MRSA/MRE), viruses and fungi. Wound odour is quickly and reliably eliminated when using SAFESEPT MAX<sup>®</sup> wound spray.

### APPLICATION:

The application of SAFESEPT MAX<sup>®</sup> wound spray is non-irritating. The affected area is sprayed with SAFESEPT MAX<sup>®</sup> wound spray by pressing the pump several times, at a short distance to fully moisten the wound. If necessary, for mechanical cleaning and debridement the application can be repeated several times. Residual SAFESEPT MAX<sup>®</sup> wound spray can be left on the wound. Dressings or bandages that are difficult to remove can be sprayed with SAFESEPT MAX<sup>®</sup> wound spray until they are wet. After a short waiting period, they can then be carefully removed. There are no restrictions regarding the frequency and duration of use. A slight chlorine odour indicates the product is working.

### APPLICATION RESTRICTIONS:

There are no known side effects of using SAFESEPT MAX<sup>®</sup> wound spray. SAFESEPT MAX<sup>®</sup> wound spray should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Interactions with other substances are not expected. During pregnancy and when breastfeeding SAFESEPT MAX<sup>®</sup> wound spray should be used with caution.

### GENERAL INFORMATION:

Not intended for infusion or injection! Only apply to external dermal wounds. Do not swallow or spray in eyes. Keep away from children. Only use intact, undamaged bottles. Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks.

### INGREDIENTS:

Water, sodium hypochlorite <0.08% (electrochemically activated mineral saline solution)

### AVAILABLE DELIVERY FORMS:

REF **988-2778** 75ml spray bottle